



VITREX® STERIHEEL® Baby CUTTING SAFETY LANCET

FOR HIGH VOLUME BLOOD FLOW WITH MINIMUM
DEPTH OF PENETRATION

- ◆ **USER SAFETY:** Blade is fully shielded before and after use. Activation button secured by safety catch.
- ◆ **USER CONVENIENCE:** Pre-loaded, button activated design.
- ◆ **PATIENT SAFETY:** Single-use only. Cannot be re-used. Sterilised by gamma irradiation. Controlled penetration depth.
- ◆ **PATIENT COMFORT:** Ultra sharp blade with high speed action securing minimal trauma and fast healing.
- ◆ **HIGH BLOOD FLOW:** The controlled movement of the blade is designed to optimise blood flow.
- ◆ **APPLICATION:** Primarily newborns and preemies. Can also be recommended for other applications where short penetration depth and/or minimal compression of tissue is important.

COLOUR CODE	SPECIFICATION	ARTICLE No	APPLICATION
Yellow	1.0 mm Depth / 2.5 mm Incision Length	333113	Newborn
Orange	0.85 mm Depth / 1.75 mm Incision Length	334113	Preemie

Shelf life 5 years



▶ **WARNING:**
Do not use if protective cap/safety cover has previously been removed or if the shield blister packing has been damaged.



▼ 1. Pull off the safety rod (Prevents premature firing)

▼ 2. Reference position for infant heel blood sampling



▼ 3. Press the device firmly against the infants heel and release the lancet



▼ 4. Collect the blood sample with a capillary tube



▼ 5. Dispose of the safety lancet in a suitable safety container

PACKAGING: BLISTER PACKED

Primary pack: 50 pcs
Shipper: 200 pcs
Pallet: 24,000 pcs