



For your safety  
and patient com-  
fort


## VITREX® STERILANCE® Lite SAFETY LANCET

Health workers the world over face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

- ◆ USER SAFETY needle is fully shielded before and after use
- ◆ USER CONVENIENCE pre-loaded, button operated design
- ◆ PATIENT SAFETY single-use only, sterilised by gamma irradiation, cannot be re-used
- ◆ PATIENT COMFORT high quality, ultra sharp tri-bevel needle, high speed penetration
- ◆ EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION Blood glucose and other in vitro diagnostic tests

COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO	APPLICATION
Yellow	18G blade	1.2 mm	383113	Special baby care and maternity units for heel puncture - neonates and newborns
Pink	26G needle	1.8 mm	384113	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	21G needle	1.8 mm	385113	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests. For paediatrics and most normal adult skin
Grey	21G needle	2.4 mm	386113	As above but for thicker adult skin.
Green	21G needle	2.8 mm	387113	Larger blood samples

Shelf life 5 years

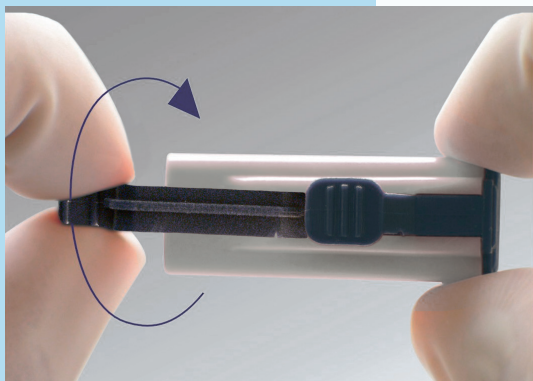

- ▶ Twist off the protective cap/safety cover (Prevents premature firing)

- ▶ **WARNING:** Do not use if protective cap/safety cover has previously been removed



- ▶ Select fingertip insertion site slightly off centre to minimise any discomfort and maximise blood flow
- ▶ Hold the lancet firmly against site and press the release button to activate the device



- ▶ Dispose of safety lancet in a suitable safety container



- ▶ Gently massage the finger in the direction of the puncture site to obtain the required volume of blood

#### PACKAGING

Primary pack:	100 pcs
Shipper:	20 x 100 primaries
Pieces per shipper:	2,000 pcs
Pallet:	96,000 pcs