




Health workers the world over face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

VITREX® STERILANCE® Press SAFETY LANCET

- ◆ USER SAFETY needle is fully shielded before and after use
- ◆ USER CONVENIENCE pre-loaded, pressure activated design
- ◆ PRESSURE ACTIVATION ensures a good blood sample every time
- ◆ PATIENT COMFORT high quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration
- ◆ EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION blood glucose and other in vitro diagnostic tests


COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO.	APPLICATION
Green	18G blade	1.8 mm	393113	Special baby care and maternity units for heel puncture - neonates and newborns
Yellow	26G needle	1.8 mm	394113	Diabetes clinics/hospital wards for blood glucose monitoring
Orange	21G needle	2.2 mm	395113	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests.
Pink	21G needle	2.8 mm	396113	As above but for thicker adult skin

Shelf life 3 years

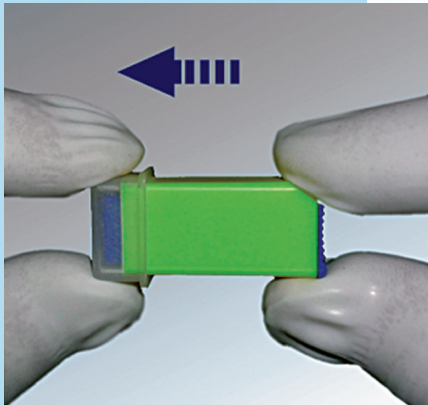

▶ **WARNING:** Do not use if protective safety cap has previously been removed

▶ Carefully pull off the protective cap

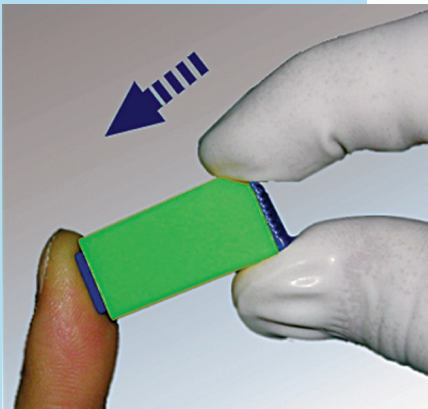

▶ Select site and press the device firmly against the site to release the lancet



▶ Dispose of safety lancet in a suitable safety container

PACKAGING

Primary pack: 100 pcs
 Shipper: 20 x 100 primaries
 Pieces per shipper: 2,000 pcs
 Pallet: 98,000 pcs