




VITREX[®] Safety Scalpels

PROTECTED DISPOSABLE SAFETY SCALPELS

- ◆ VITREX[®] Safety Scalpels are designed to minimize the risk of cross contamination without compromising neither userfriendliness nor sharpness.
- ◆ The scalpels are available with all common handles and blades.

How to use Safety Scalpels

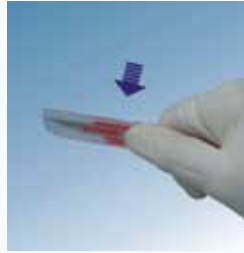

Open

- ▶ Press button and pull back to expose blade
- ▶ **Don't push forward**



Ready for use

- ▶ When you hear a click, the device is fully opened and ready for use.




Close for re-use

- ▶ Push cover forward carefully until you hear a light click.



Safety Lock before disposal

- ▶ Push cover further forward until you hear a loud click.

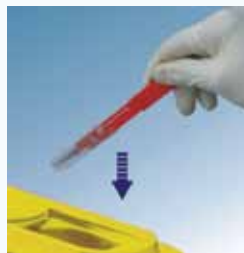

Disposal

- ▶ Dispose of the Safety Scalpel in a suitable safety container

