



For your safety and patient comfort

## VITREX® STERILANCE® Lite II SAFETY LANCET

- ◆ USER SAFETY: Needle is fully shielded before and after use
- ◆ USER CONVENIENCE: Pre-loaded, button operated design
- ◆ PATIENT SAFETY: Single-use only, sterilised by gamma irradiation, cannot be re-used
- ◆ PATIENT COMFORT: High quality, ultra sharp tri-bevel needle, high speed penetration
- ◆ EFFICACY: Variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION: Blood glucose and other in vitro diagnostic tests

COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO	APPLICATION
Purple	28G needle	1.8mm	3741-13	Diabetes clinics/hospital wards for blood glucose monitoring
Yellow	26G needle	1.8mm	3751-13	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	26G needle	2.4mm	3761-13	As above but for thicker adult skin.
Orange	21G needle	1.8mm	3771-13	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests. For paediatrics and most normal adult skin
Pink	21G needle	2.4mm	3781-13	As above but for thicker adult skin.

Shelf life 5 years



- ▶ Twist off the protective cap

- ▶ **WARNING:** Do not use if protective cap has previously been removed



- ▶ Select fingertip insertion site slightly off centre to minimise any discomfort and maximise blood flow
- ▶ Hold the lancet firmly against site and press the release button to activate the device



- ▶ Dispose of safety lancet in a suitable safety container



- ▶ Gently massage the finger in the direction of the puncture site to obtain the required volume of blood

#### PACKAGING

Primary pack:	100 pcs
Shipper:	20 x 100 primaries
Pieces per shipper:	2,000 pcs
Pallet	84.000 pcs