



For your safety
and patient com-
fort


VITREX[®] STERILANCE[®] Lite SAFETY LANCET

Health workers the world over face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

- ◆ USER SAFETY needle is fully shielded before and after use
- ◆ USER CONVENIENCE pre-loaded, button operated design
- ◆ PATIENT SAFETY single-use only, sterilised by gamma irradiation, cannot be re-used
- ◆ PATIENT COMFORT high quality, ultra sharp tri-bevel needle, high speed penetration
- ◆ EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION Blood glucose and other in vitro diagnostic tests

COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO	APPLICATION
Yellow	18G blade	1.2mm	3831-13	Special baby care and maternity units for heel puncture - neonates and newborns
Pink	26G needle	1.8mm	3841-13	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	21G needle	1.8mm	3851-13	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests. For paediatrics and most normal adult skin
Grey	21G needle	2.4mm	3861-13	As above but for thicker adult skin.
Green	21G needle	2.8mm	3871-13	Larger blood samples

Shelf life 5 years

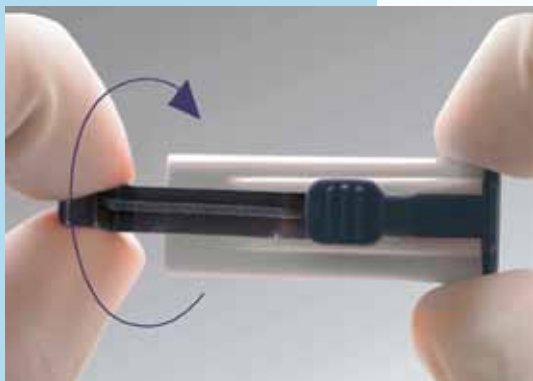

- ▶ Twist off the protective cap/safety cover (Prevents premature firing)


- ▶ **WARNING: Do not use if protective cap/safety cover has previously been removed**



- ▶ Select fingertip insertion site slightly off centre to minimise any discomfort and maximise blood flow
- ▶ Hold the lancet firmly against site and press the release button to activate the device



- ▶ Dispose of safety lancet in a suitable safety container

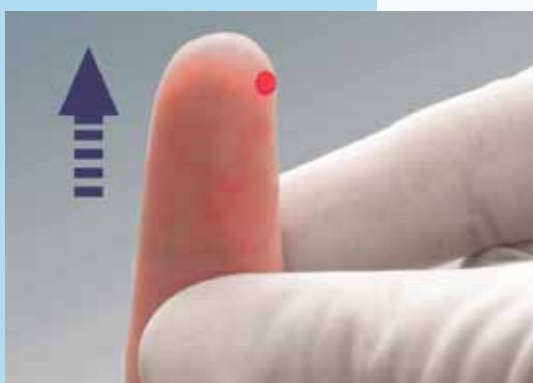

- ▶ Gently massage the finger in the direction of the puncture site to obtain the required volume of blood

PACKAGING

Primary pack:	100 pcs
Shipper:	20 x 100 primaries
Pieces per shipper:	2,000 pcs
Pallet	96.000 pcs