




Health workers the world over face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

VITREX® STERILANCE® Press SAFETY LANCET

- ◆ USER SAFETY needle is fully shielded before and after use
- ◆ USER CONVENIENCE pre-loaded, pressure activated design
- ◆ PRESSURE ACTIVATION ensures a good blood sample every time
- ◆ PATIENT COMFORT high quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration
- ◆ EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements
- ◆ APPLICATION blood glucose and other in vitro diagnostic tests


COLOUR CODE	CONFIGURATION	PENETRATION DEPTH	ARTICLE NO	APPLICATION
Green	18G blade	1.8mm	3931-13	Special baby care and maternity units for heel puncture - neonates and newborns
Yellow	26G needle	1.8mm	3941-13	Diabetes clinics/hospital wards for blood glucose monitoring
Orange	21G needle	2.2mm	3951-13	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests.
Pink	21G needle	2.8mm	3961-13	As above but for thicker adult skin

Shelf life 3 years

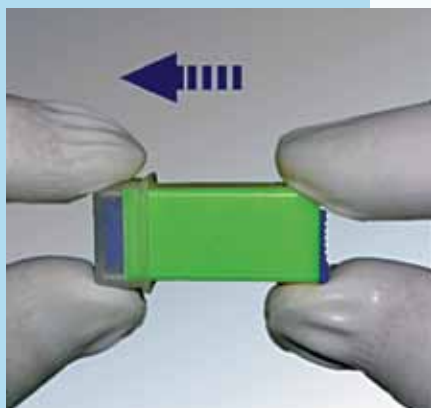

- ▶ **WARNING:** Do not use if protective safety cap has previously been removed

- ▶ Carefully pull off the protective cap

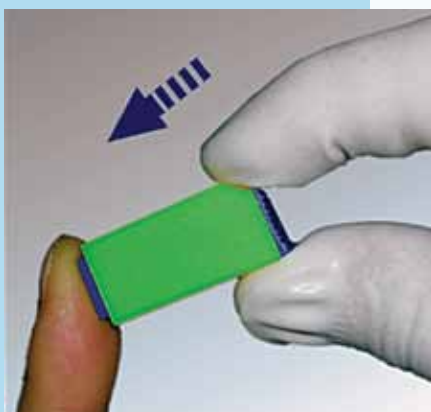

- ▶ Select site and press the device firmly against the site to release the lancet



- ▶ Dispose of safety lancet in a suitable safety container

PACKAGING

Primary pack:	100 pcs
Shipper:	20 x 100 primaries
Pieces per shipper:	2,000 pcs
Pallet	98.000 pcs