


Vitrex Ultrasafe Sterile Safety Lancet


Instruction for use




**EUROPEAN
NEEDLES**
AT IT'S BEST
FOR COMFORT & SAFETY

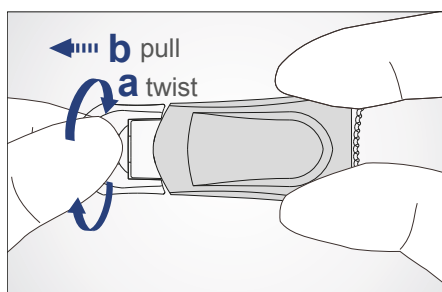

1. Twist and pull off the lancet cap.

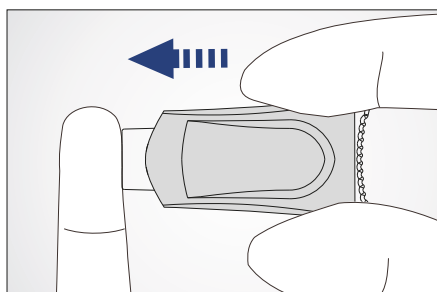

2. Press the lancet body firmly against chosen site to activate the device.

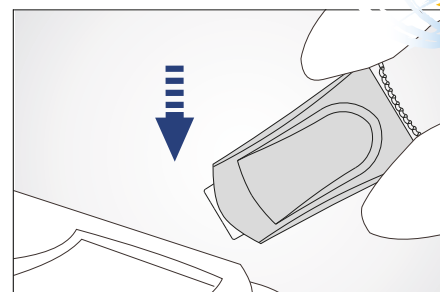

3. Dispose in a suitable sharps container.


PRODUCT:

Vitrex Ultrasafe Safety Lancets are used to carry out skin punctures to collect micro-samples of capillary blood.

- Contact activated safety lancet.
- European high-quality silicone coated needle for softer penetration.
- Trigger position indicates the before and after use.
- Easy twist-off cap design.
- Sterile, single use and disposable.
- Ergonomic case and user-friendly design.
- Easy to use, 2-step operation and no charging.
- Safety, after use the lancet retracts inside the case, avoiding accidents.
- Exclusive to Vitrex.

The dimensions:

- Length: 29.2 ± 0.3 mm
- Width: 17.4 ± 0.3 mm
- Bottom: 40.4 ± 1.0 mm

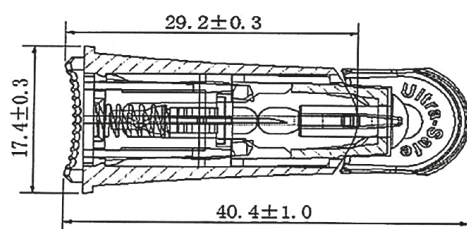

Warning:

Do not use, if lancet cap has been previously removed from lancet. Discard used lancet into a sharps container in accordance with facility guidelines, national and state regulations.

TECHNICAL SPECIFICATIONS:

| Colour Code | Configuration | Article No. |
|-------------|---------------------------------|-------------|
| Green | 17G / 2.00 mm Penetration depth | 393413 |
| Blue | 21G / 1.80 mm Penetration depth | 397413 |
| Orange | 21G / 2.20 mm Penetration depth | 395413 |
| Pink | 21G / 2.80 mm Penetration depth | 396413 |
| Yellow | 26G / 1.80 mm Penetration depth | 394413 |
| Purple | 28G / 1.80 mm Penetration depth | 398413 |

Shelf life 5 years

