



## Recommended use

### 1. Variable penetration

Dial the appropriate depth on the device between 1 (min.) and 5 (max.).

### 2. Simple and quick activation and reactivation

Pull back the grey slider to activate the mechanism.

### 3. Instant operation

Depress the grey button to operate the device.

### 4. Lancet removal

Simple and reliable lancet ejection.

## Vitrex Compact Eject

Vitrex Compact Eject is a lancing device with a push button ejection. The device is clear adjustable with clear fixed end caps designed for Alternate Site Testing (AST).

### VERSATILE

Suitable for use with Vitrex Soft Lancets and other most commonly used lancet brands.

### VARIABLE PENETRATION

Nine depth settings from 1 to 5.

### CONVENIENT

Simple and quick activation, reactivation and non-touch lancet removal.

## Technical Specifications

Product	Description	Packaging	Article No.
Vitrex Compact Eject	9-step adjustable depth	Clear plastic sleeve only	369613

Shelf life 5 years

