



Vitrex Octaheel Baby

Sterile Cutting Safety lancets

Vitrex Octaheel Baby lancets are intended for high volume blood flow with minimum depth of penetration. The safety lancets are primarily applicable for newborns and preemies. However, they can also be recommended for other applications, where short penetration depth and/or minimal compression of tissue is important.

Vitrex Octaheel Baby lancet is the latest edition offering improved ergonomics.



ACTIVATION BUTTON: Secured by safety tab.

NO CHOCKING HAZARD: Non-detachable safety tab.

USER SAFETY: Blade is fully shielded before and after use.

EASIER HANDLING: Anti-slip texture and one hand operation.

USER CONVENIENCE: Pre-loaded, button activated design.

PATIENT SAFETY: Single-use only. Sterilised by gamma irradiation.

PATIENT COMFORT: Ultra sharp blade with high speed cutting securing minimal trauma and fast healing.

NARROWED LANCING SLOT: Easier positioning.

1

2

3

4






5

LANCETS

Recommended use

1. Release the trigger lock mechanism by pushing down the safety tab.
2. Reference position for infant heel blood sampling.
3. Raise the baby's foot to a suitable position, place the blade slot flush against the heel site and activate the lancet.
4. Collect the blood sample with a capillary tube.
5. Dispose the safety lancet in a suitable safety container.

Technical Specifications

Colour Code	Specification	Article No.	Indicative use
Green 	0.65 mm Depth / 1.40 mm Incision Length	335313	Micro-Premie
Orange 	0.85 mm Depth / 1.75 mm Incision Length	334313	Premie
Yellow 	1.00 mm Depth / 2.50 mm Incision Length	333313	Newborn
Pink 	1.14 mm Depth / 2.80 mm Incision Length	332313	Newborn Plus
Blue 	2.00 mm Depth / 3.00 mm Incision Length	331313	Toddler

Shelf life 5 years



Transit packaging

Pallet

Packaging:

Blister Packed

Primary packaging: 50 pcs.
Transit packaging: 200 pcs.
Pallet: 23,200 pcs.