




## How to Use

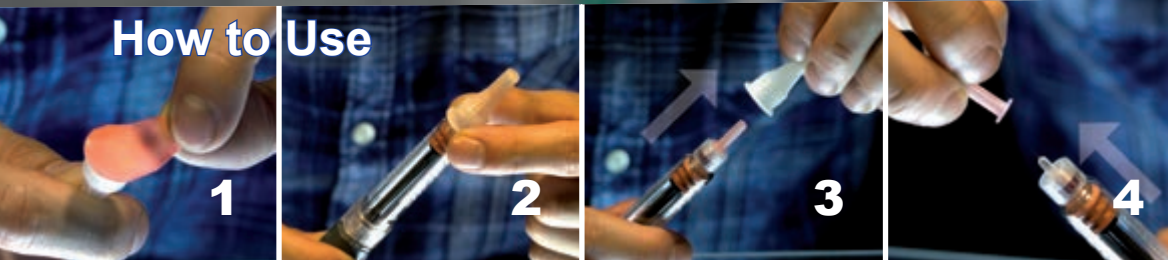

1 Pull the paper tab off the pen needle.

2 Screw needle onto the end of the insulin pen.

3 Remove outer needle cover.

4 Remove inner needle cover to expose the needle.

Your pen is now ready for use.

## Medoject Fine® Pen Needles

Medoject Fine Pen Needles are made of polypropylene hub and stainless steel tube. They are intended for Insulin application via most type A Insulin pens. The needles are sterile and individually packed.

### COMFORTABLE INJECTION

European, premium needle quality, triple beveled tip and precision lancet point grinding.

### THIN WALL TECHNOLOGY

Ensures optimal insulin flow.

### SILICONIZED NEEDLE

Minimizes irritation and gives a smooth penetration.

### SHORT NEEDLES

Reduce risk of a painful injection into the muscle.

### VERSATILE

Fit most commonly used insulin pens.

### COLOUR-CODING

Simple identification of different needle lengths.

## Technical Specifications

Colour Code	Specification	Article No.
Yellow	29G x 10 mm	CHPN2910
Pink	29G x 12.7 mm	CHPN29127
Dark blue	30G x 6 mm	CHPN3006
Light blue	30G x 8 mm	CHPN3008
Green	31G x 4 mm	CHPN3104
Dark violet	31G x 5 mm	CHPN3105
Dark blue	31G x 6 mm	CHPN3106
Light blue	31G x 8 mm	CHPN3108
Green	32G x 4 mm	CHPN3204
Dark violet	32G x 5 mm	CHPN3205
Dark blue	32G x 6 mm	CHPN3206

MEDOJECT Fine Pen Needles are available in many sizes. Ask your diabetes educator or pharmacist for advice.


**Shelf life 5 years**



**Packaging:**  
 Primary packaging: 100 pcs.  
 Transit packaging: 2,000 pcs.  
 Pallet: 160,000 pcs.



5

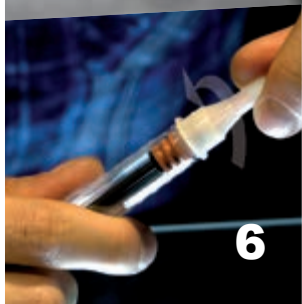

6



7



8



MySharps  
- new container for used  
insulin pen needles.

Place the outer needle cover  
over the needle.

Twist off the needle from  
the insulin pen.

Discard the used pen needle  
into a safety container.

## Vitrex Pen Needles

### COMFORTABLE INJECTION

Premium needle quality and triple beveled tip.

### THIN WALL TECHNOLOGY

Ensures optimal insulin flow.

### SILICONIZED NEEDLE

Minimizes irritation and reduces penetration forces.

### VERSATILE

Fit most commonly used insulin pens.

### COLOUR-CODING

Simple identification of different needle sizes.

## Technical Specifications

Colour Code	Specification	Article No.
Rose	29G x 12 mm	530313
Dark Blue	30G x 6 mm	530413
Turquoise	30G x 8 mm	530113
Grey	31G x 5 mm	530713
Orange	31G x 6 mm	530513
Green	31G x 8 mm	530613
Peach	32G x 4 mm	530213

Vitrex Pen Needles are available in many sizes.  
Ask your diabetes educator or pharmacist for advice.

**Shelf life 5 years**



Transit packaging

Pallet

**Packaging:**  
Primary packaging: 100 pcs.  
Transit packaging: 2,000 pcs.  
Pallet: 128,000 pcs.